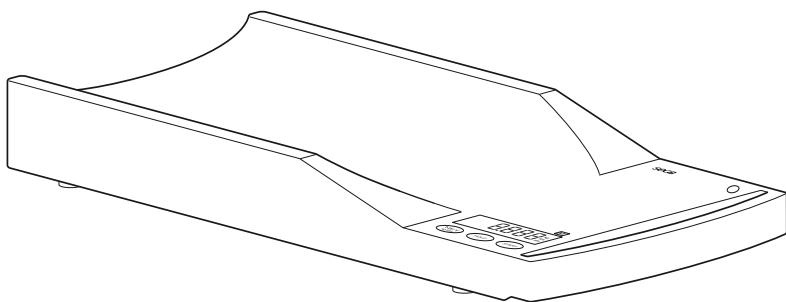
**seca 336kg**

|            |   |            |
|------------|---|------------|
| <b>D</b>   | <b><i>Bedienungsanleitung und Garantieerklärung</i></b>         | <b>3</b>   |
| <b>GB</b>  | <b><i>Instruction manual and guarantee</i></b>                  | <b>11</b>  |
| <b>F</b>   | <b><i>Mode d'emploi et garantie</i></b>                         | <b>19</b>  |
| <b>I</b>   | <b><i>Manuale di istruzioni e garanzia</i></b>                  | <b>27</b>  |
| <b>E</b>   | <b><i>Manual de instrucciones y garantía</i></b>                | <b>35</b>  |
| <b>DK</b>  | <b><i>Betjeningsvejledning og garantibevis</i></b>              | <b>43</b>  |
| <b>S</b>   | <b><i>Bruksanvisning och garanti</i></b>                        | <b>51</b>  |
| <b>N</b>   | <b><i>Bruksanvisning og garantierklæring</i></b>                | <b>59</b>  |
| <b>FIN</b> | <b><i>Käyttöohje ja takuu</i></b>                               | <b>67</b>  |
| <b>NL</b>  | <b><i>Bedieningshandleiding en garantieverklaring</i></b>       | <b>75</b>  |
| <b>P</b>   | <b><i>Instruções de utilização e declaração de garantia</i></b> | <b>83</b>  |
| <b>GR</b>  | <b><i>Οδηγίες χειρισμού και εγγύηση</i></b>                     | <b>91</b>  |
| <b>PL</b>  | <b><i>Instrukcja obsługi i deklaracja gwarancji</i></b>         | <b>101</b> |

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---

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# Full certification

---

In seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes.

seca products comply with European directives, standards and national laws. With seca, you are buying the future.

**M**

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol are approved for medical purposes in the European Community. seca scales with this symbol meet the stringent quality and technical requirements placed on scales which can be calibrated.

Scales carrying this symbol are calibrated in accordance with Precision Class III of EC Directive 90/384/EEC.

Products carrying this symbol comply with the following directives and standards:

1. Directive 90/384/EEC governing non-automatic weighing instruments
2. Directive 93/42/EEC governing medical devices
3. DIN EN 45501 governing metrological aspects of non-automatic scales

seca's professionalism has also been officially recognized. The certificate from TÜV Product Service, the body responsible for medical devices, confirms that as a manufacturer of medical devices, seca rigorously complies with stringent legal requirements. seca's quality assurance system covers the design, development, production, sales and service of medical scales and measuring systems.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

# 1. Congratulations!

---

In the **seca 336** electronic baby scale you have acquired a highly-accurate yet simultaneously sturdy piece of equipment.

seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

The **seca 336** electronic baby scale is designed for use in a doctor's surgery or hospital. It serves both to establish birth weight and to check weight gain during the growth process.

The scales are a Class I medical product, and have been calibrated in accordance with the precision class III.

## 2. Safety

---

- Follow the instructions in the instructions for use.
- Keep the operating instructions and the declaration of conformity in a safe place.
- Never leave a baby lying on the scale unsupervised.
- Ensure that the scale is standing firmly on a smooth, level surface.
- Do not drop the scale or subject it to violent shocks.
- When using the scale with a mains unit, ensure that the supply cable is routed in such a way as to exclude any type of tripping hazard.
- Use only the type of battery stated (see "Battery and mains (option)" on page 13).
- Have scale serviced and re-calibrated on a regular basis (see "Servicing/recalibration" on page 17).
- Have repairs carried out only by authorised persons.

## 3. Before you start...

---

### Power supply

The scale is powered by batteries or a mains unit (option). The LC display has a low power consumption – 4000 weighing operations can be performed with one set of batteries. The  symbol first appears when battery level starts dropping, and **bAtE** appears in the display when the batteries have discharged further.

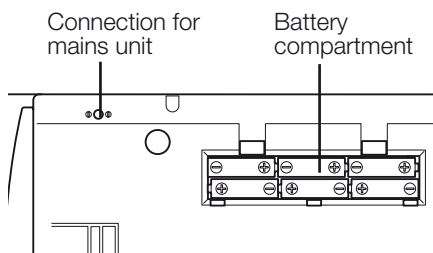
## Battery and mains (option)

If  or *batt* appear in the display, you should change the batteries. You need 6 1.5 V type AA, R6 or LR6 round-cell batteries.

- Carefully turn over the scale so that the base of the scale is accessible.
- Press the latches of the battery compartment towards the centre of the base and take off the cover.
- Place the new batteries in the battery compartment.

### Check that polarity is correct!

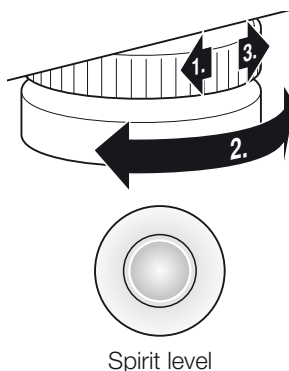
- Close the cover and then turn the scale back up the right way.

## 4. Setting up the scale

The scale is fully assembled on delivery.

- Remove the packaging and place the scale on a firm, level surface.
- Undo the locknuts using the four foot screws (1).
- Align the scale by undoing the foot screws. The air bubble in the spirit level must be right in the centre of the circle (2).
- Then re-tighten the locknuts (3).
- Ensure that only the feet of the scale are in contact with the floor. The scale may not be in contact at any other point.

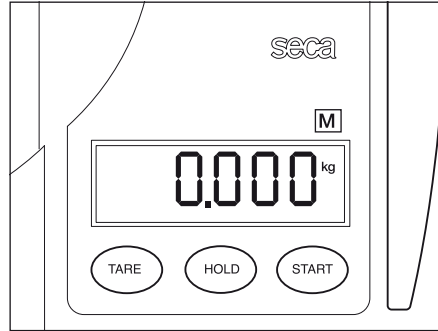
# 5. Operation

---

## Correct weighing

- Press the Start key with no load on the scale.  
**SECA**, **88888** and **0.000** appear consecutively in the display. The scale is then automatically set to zero and ready for use.
- Place the baby on the scale.
- Read the weighing result off the digital display.
- To switch off the scale, press the Start key again.

If the scale is operated with a mains unit, the  $\Psi$  symbol appears in the display.

## Automatic switch-off/economy mode

To save energy, a scale with no load on it switches off automatically after approx. 5 minutes. In mains operation, the scales remain switched on permanently.

## Taring weight display (TARE)

The tare function is useful if an additional weight (such as cloths or a pad on the weighing surface) are to be ignored in the weighing result. Proceed as follows:

- Place the additional weight on the scale and press the button **TARE**.  
The note "NET" appears.
- Wait until the display stops flashing and shows **0.000**.
- You can now perform as many weighing operations as you like with this additional weight.
- TARA is deactivated by pressing the button **TARE** once again, or by switching the scales off.

## Important!

The maximum additional weight is 10 kg. The maximum weight which can be displayed is reduced by the weight of the objects already placed on the scale.

## Storing the weight value (HOLD)

- Press the **HOLD** key whilst the load is on the scale.  
The display flashes until a stable weight has been measured. The display is then frozen until the next weighing operation takes place.
- “HOLD” and  appears in the digital display. The triangle with the exclamation mark indicates that this frozen display is an additional facility which is not subject to calibration.
- The HOLD function can be switched off by pressing the **HOLD** key again.  
 appears in the display if the scale has no load on it, otherwise the current weight. The “HOLD” display vanishes.

### Note:

The HOLD function can also be used if the TARE function has been executed beforehand.

## Checking that the contents of the calibration counter are correct

This seca scale is calibrated. Calibration may be performed only by authorised centres. To ensure this, the scale is fitted with a calibration counter which records any change to data relevant to calibration. If you want to check whether the scale is properly calibrated, proceed as follows:

- If necessary, switch off the scale.
- Keep any key depressed and start the scale.
- The current contents of the calibration counter are shown on the display for a few seconds.
- Compare the output contents of the calibration counter with the number shown on the calibration counter sticker.

For valid calibration, both numbers must match. If sticker and calibration counter do not match, then re-calibration is required. Contact your service agent or seca Customer Service.

Both numbers must match.

If a supplementary calibration is necessary, then the supplementary calibration mark illustrated opposite is used to record the calibration counter position instead of the seca calibration mark shown above. This mark is secured with an additional seal by the person authorised to perform the supplementary calibration. The supplementary calibration mark is available from seca Customer Service under the number 14-05-01-886.

## 6. Cleaning

---

Clean the tray and the housing as required with a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

## 7. What do I do if...

---

### ... no weight is displayed with a load on the scale?

- Is the scale switched on?
- Check the mains unit connection.

### ... 0.0000 does not appear before weighing?

- Press the Start key again - there must not be any load on the scale.

### ... one segment is illuminated permanently or not at all?

- That point has a fault. Inform your service agent.

### ... the display shows ?

- Battery voltage is dropping. Charge the rechargeable batteries in the next few days.

### ... bAtE appears in the display?

- The rechargeable battery is so severely discharged that weighing is no longer possible. Recharge the battery.

### ... 5tOP appears in the display?

- Maximum load has been exceeded.

### ... the display LErrP appears?

- The ambient temperature of the scale is too high or too low. Place the scale in an ambient temperature between +10 °C and +40 °C. Wait about 15 minutes for the scale to adapt to the ambient temperature and then weigh again.

### ... Er:0 1 1: appears in the display?

- The scale is overloaded or the load is unevenly distributed. Take the load off the scale or distribute weight more evenly and restart scale.

... **Er:0 12**: appears in the display?

- The scale was switched on with too great a load on it. Remove load and restart.

## 8. Servicing/recalibration

---

Have the scale recalibrated by authorised persons in accordance with national legal requirements. The year of initial calibration can be found behind the CE symbol on the rating plate above the number of the appointed office, 0109 (Hessen Weights & Measures Office). Recalibration is essential if one or more security marks are damaged or the calibration counter does not match the number on the applicable calibration counter mark (cf. page 15).

We recommend having your scale serviced by your local service agent before having it recalibrated. seca Customer Service will be pleased to give you further assistance.

## 9. Technical data

---

### Dimensions

**length:** 300 mm

**width:** 640 mm

**height:** 105 mm

**Weight** approx. 3.5 kg

**Temperature range** +10 °C to +40 °C

**Maximum load** 15 kg

**Graduations** 10 g

**Calibration as per Directive 90/384/EEC** Class III medical approval

### Accuracy at first calibration

up to 5 kg: ± 5 g

5 kg up to 15 kg: ± 10 g

### Medical product as per Directive 93/42/EEC

Class I

**Tare range** up to 10 kg / 22 lbs

**Height of figures** 20 mm

**Power supply** Battery/  
(option mains unit)

**Power consumption** approx. 25 mA

# 10. Accessories

---

## Model dependent mains unit

Plug-in mains unit, Euro: 230V~ / 50Hz / 12V= / 150mA

seca no.: 68-32-10-252

Switchmode power adapter, 100-240V~ / 50-60Hz / 12V= / 0.5A

seca no. 68-32-10-265

### Attention!

Use only seca original plug-in mains units with 9 V or controlled 12 V output voltage. Commercially-available mains units may supply a higher voltage than stated and thus lead to damage to the scale.

## Recalibration

**Supplementary calibration mark**      seca spare part no.: 14-05-01-886

# 11. Disposal

---

## Disposing of the device

Do not dispose of the device in domestic waste. The device must be disposed of properly as electronic waste. Follow the national regulations which apply in your case. For further information, contact our service department at: [service@seca.com](mailto:service@seca.com)

## Batteries

Do not throw used batteries away in domestic waste. Dispose of batteries at collection points in the vicinity. When buying new batteries, select those low in harmful substances and containing no mercury (Hg), cadmium (Cd) or lead (Pb).

# 12. Warranty

---

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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