Otto Bock[®]

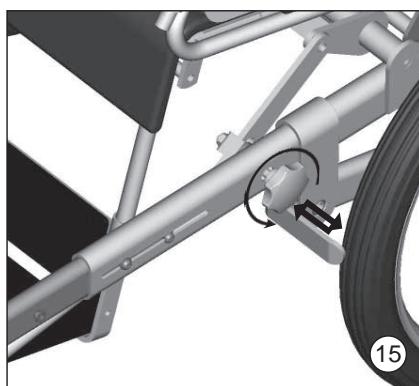
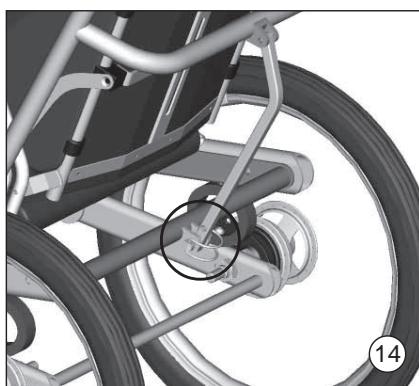
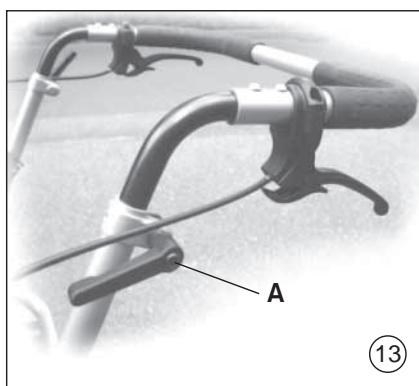
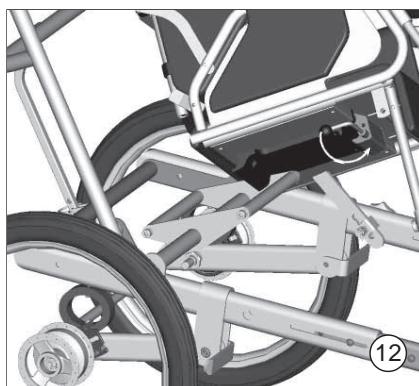
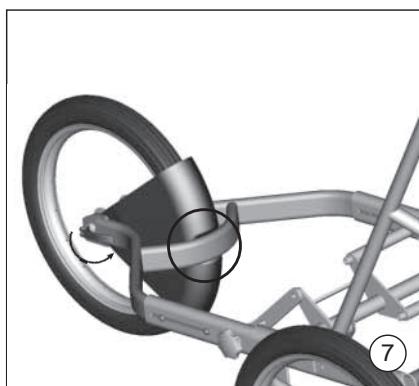
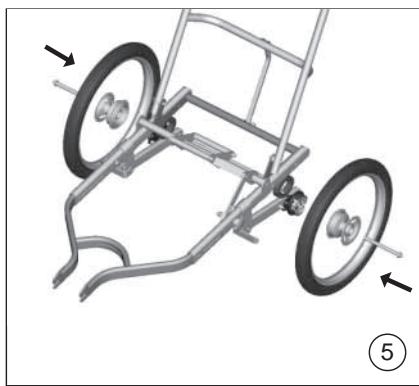
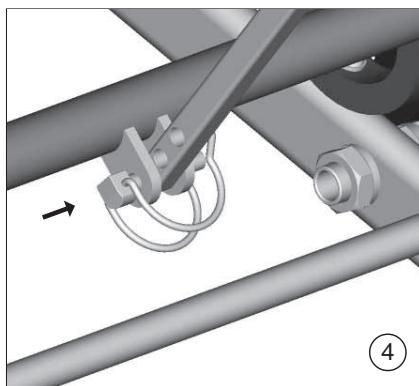
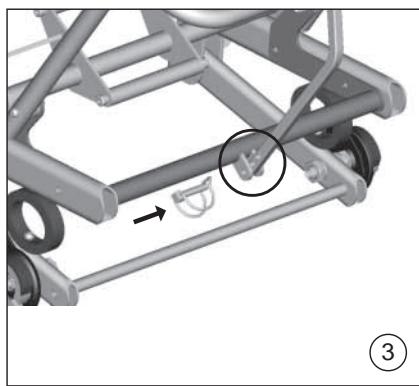
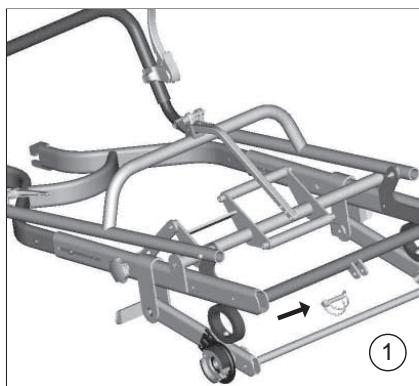
QUALITY FOR LIFE

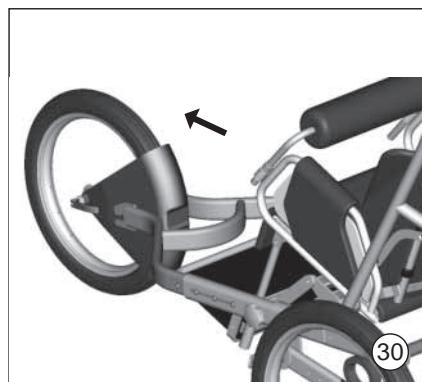
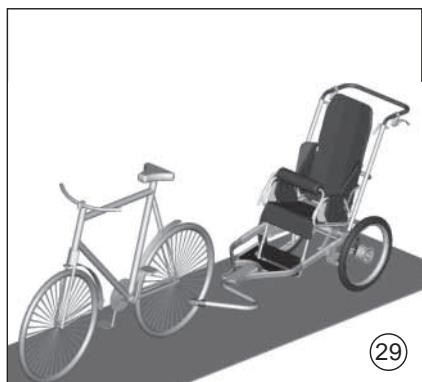
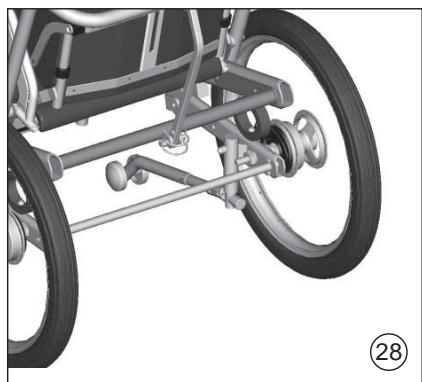
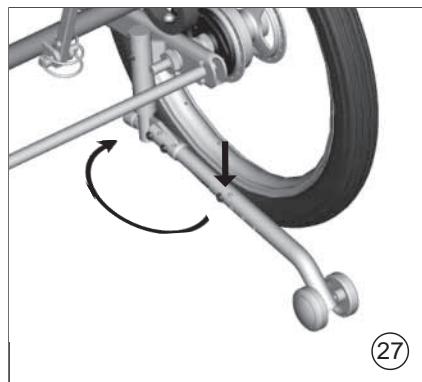
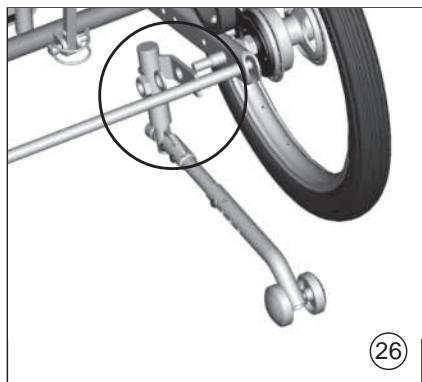
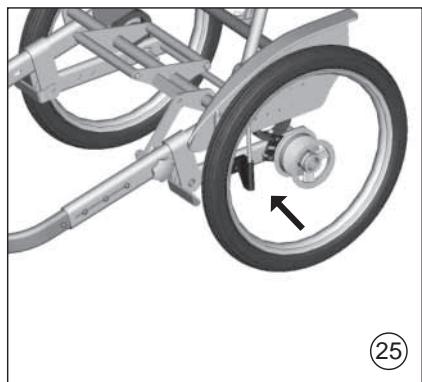
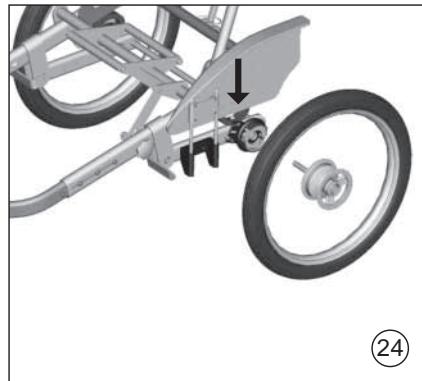
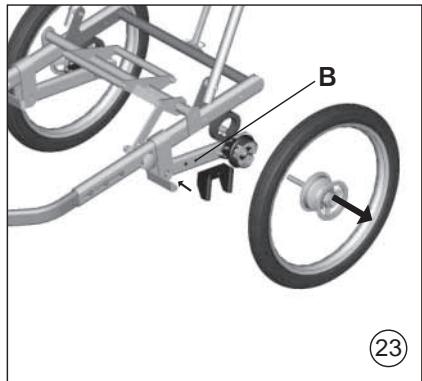
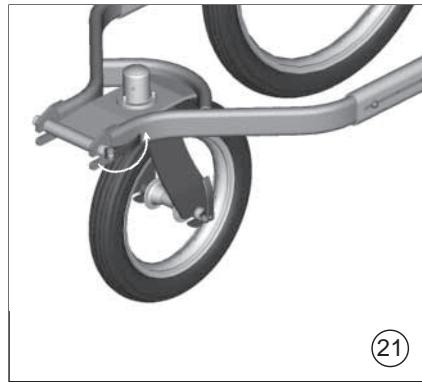
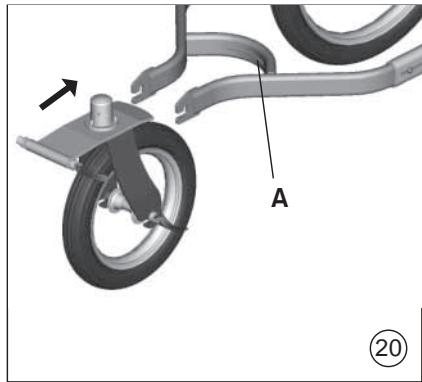
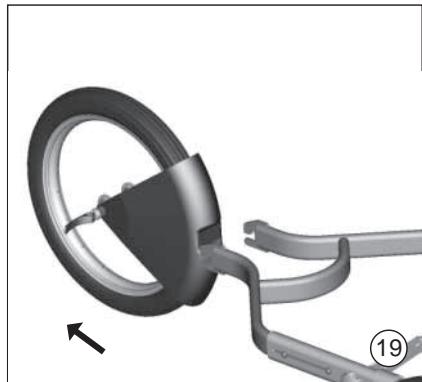
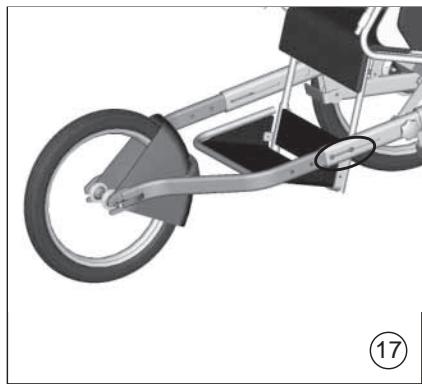
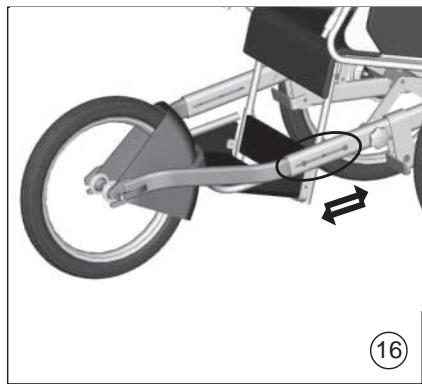
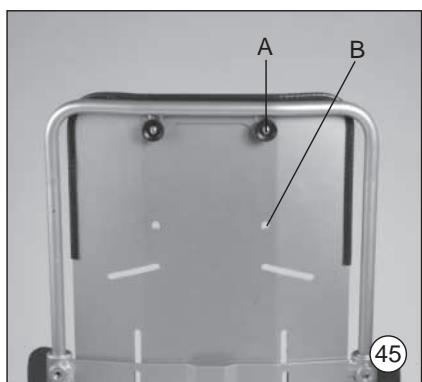
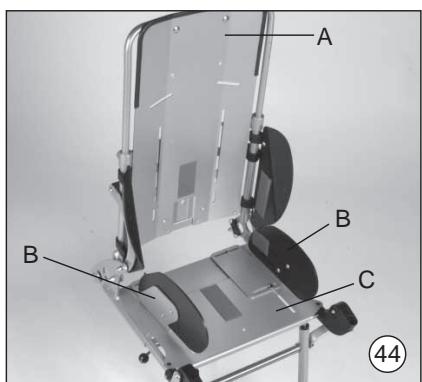
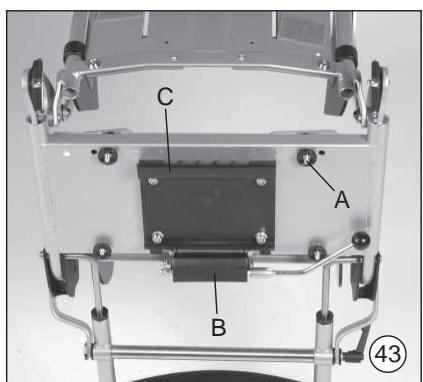
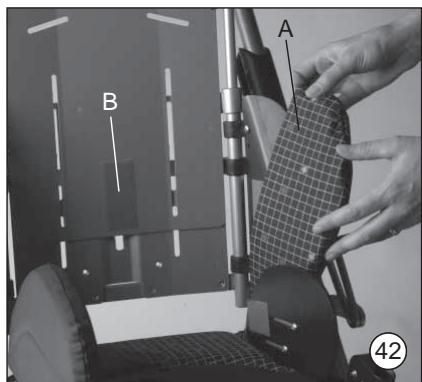
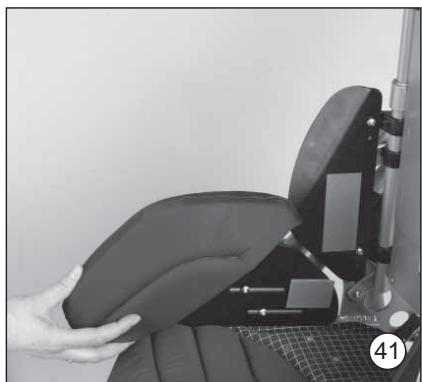
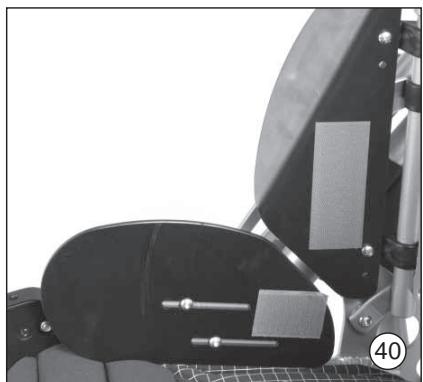
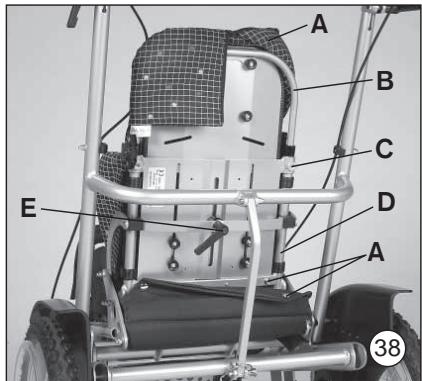
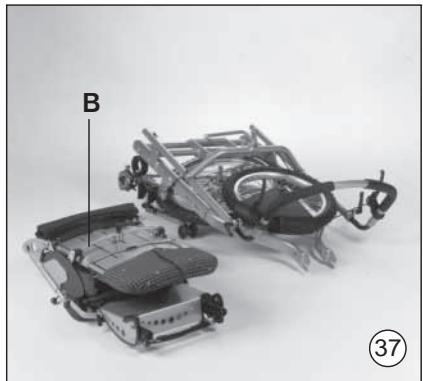
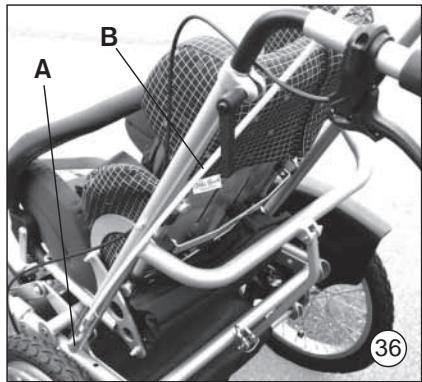
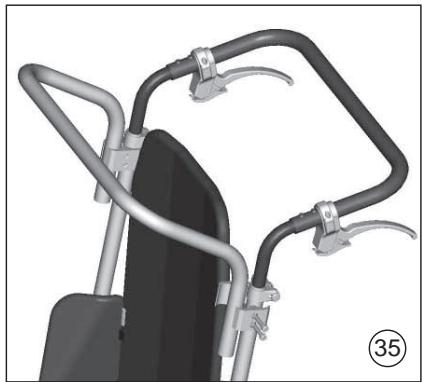
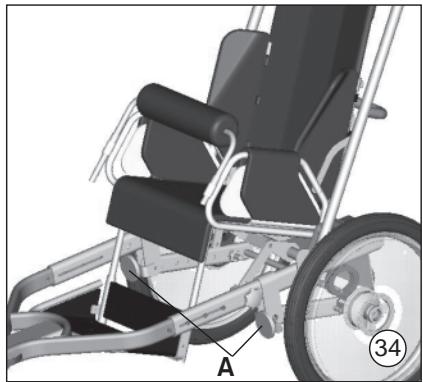
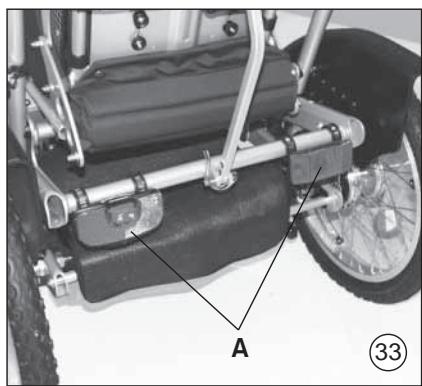
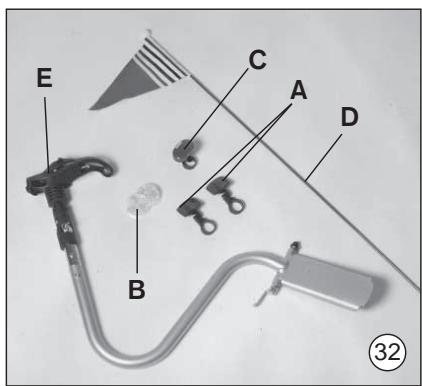
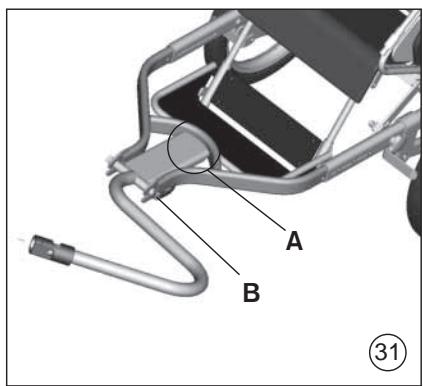
CE

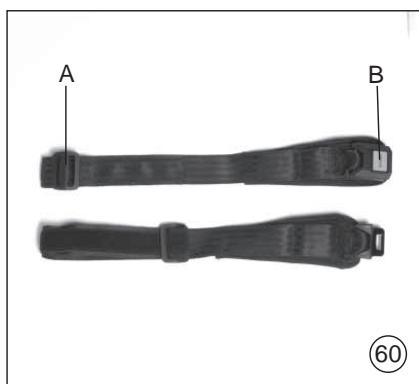
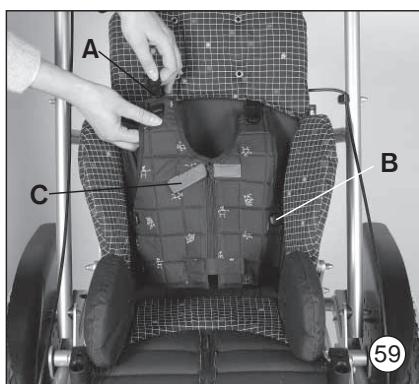
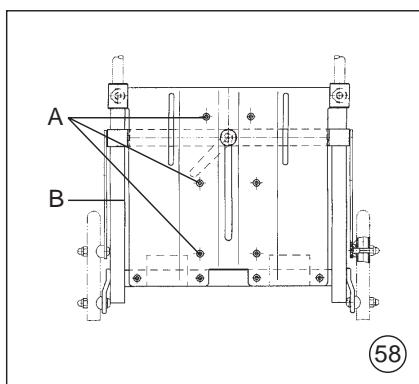
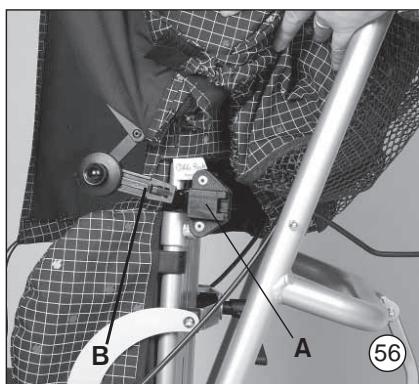
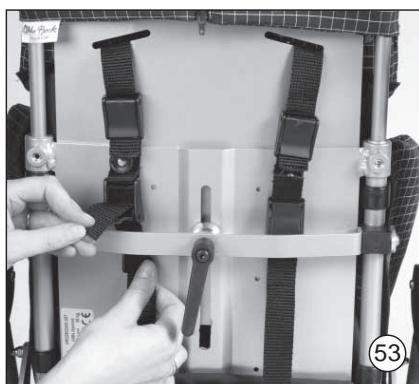
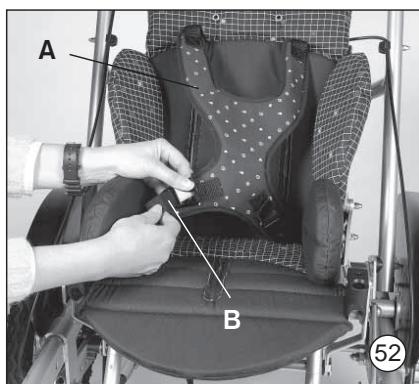
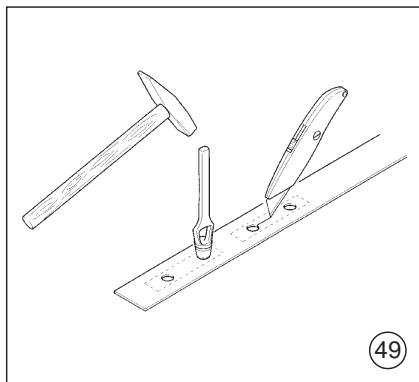
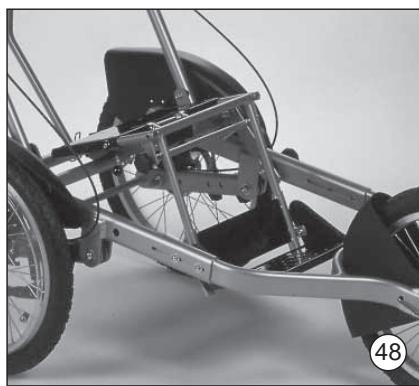
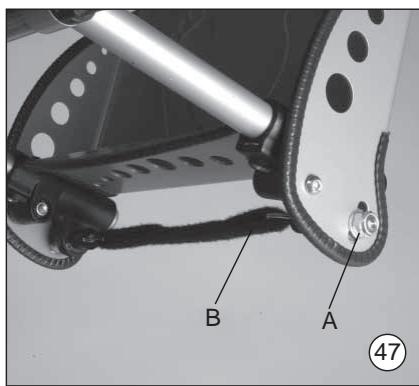
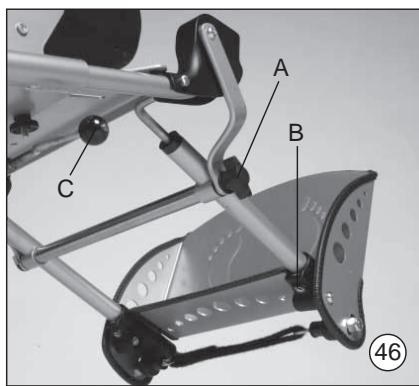
Kimba Cross

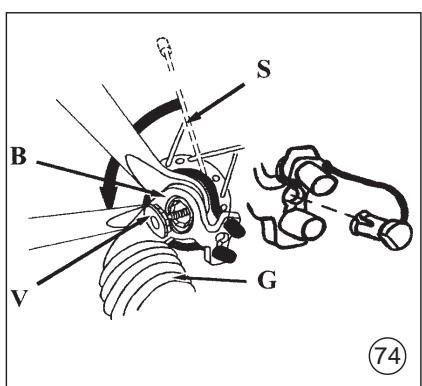
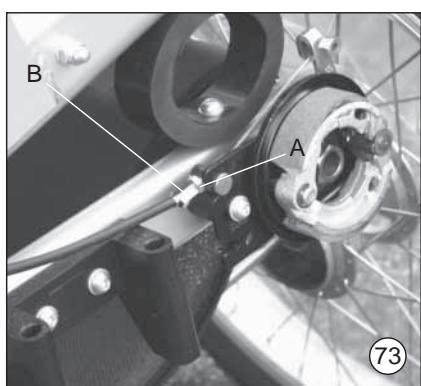
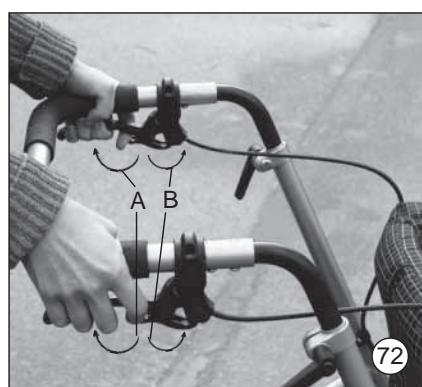
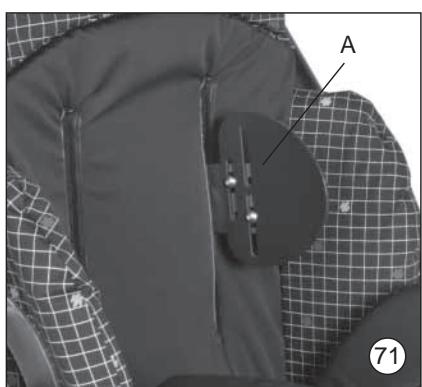
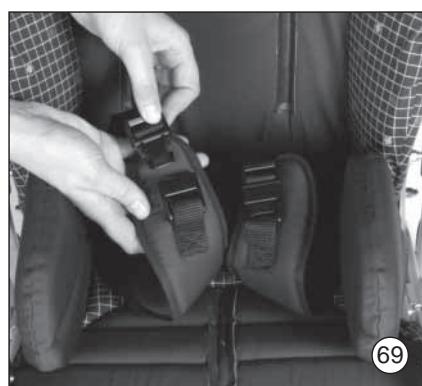
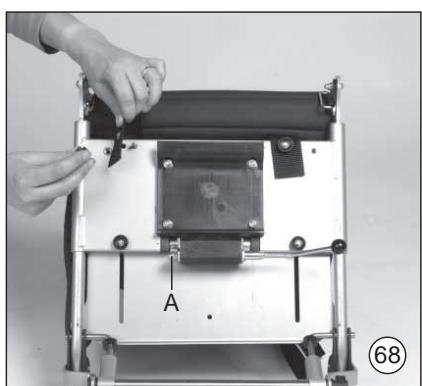
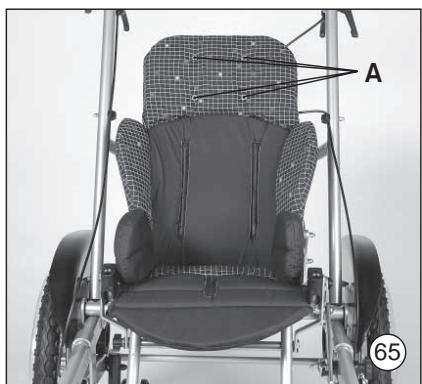
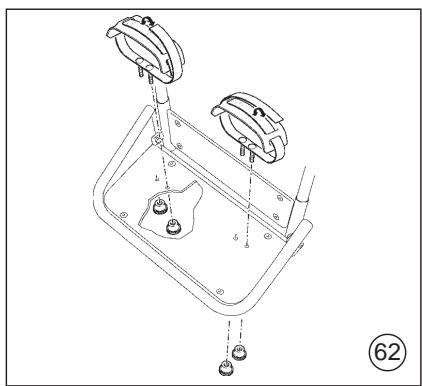
(D) <i>Bedienungsanleitung</i>	7
(GB USA) <i>Instruction for use</i>	19
(NL) <i>Gebruiksaanwijzing</i>	30

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**Instructions for Use for Kimba Cross –
Pediatric Postural Positioning System/Mobility Base for Seating Shells**

Contents	Page
Preface	20
Application – Indications	20
Declaration of Conformity	20
Service	20
Safety Instructions	20
Unpacking and Setting up	22
Outdoor Mobility Base	22
Accessories for Outdoor Mobility Base	23
Swivelling front wheel	23
Splash guard for rear wheels/Clothing protectors	23
Anti-tipper	23
Storage bag	23
Spoke protectors	23
Use as a Bicycle Trailer	23
Kimba Multifunctional Seating Unit	24
Seating Shell Interfaces	25
Accessories for Kimba Multifunctional Seating Unit:	25
Abductor	25
Armrest	25
Pelvic band	25
Chest and shoulder pad	25
Canopy	26
Fixation vest	26
Belts: Lap belt, four-point belt, foot straps	26
Grab rail and grab rail padding	26
Headrests	27
Rain cape	27
Slipsacks	27
Groin strap	27
Tray	27
Thoracic supports	27
Using your Kimba Cross for transportation in Motor Vehicles	28
Instructions for Maintenance and Cle*aning	28
Warranty	28
Technical Data	29

Preface

The Kimba Cross mobility base / pediatric postural positioning system is a quality product offering versatile use in everyday life.

Before using your Kimba Cross please read sections: "Unpacking and Setting up" and "Safety Instructions". Please keep these instructions in a safe place.

The section "Accessories" presents additional parts for the Kimba Cross, which may extend the range of application and increase the comfort.

If you have any further questions, or if you have any problems, please contact your healthcare provider.

The design, as described in these Instructions for Use, is subject to technical alterations without notice.

Application – Indications

(Source: Catalog of aids and list of aids of the health insurance funds, FRG)

The prescription of a pediatric postural positioning system is generally indicated for the severely disabled, especially for children and young people who are unable to walk, need to be transported and are unable to operate a wheelchair themselves.

Inability to walk or very serious walking impediment due to:

- Palsies
- Paralysis
- Loss of limbs
- Defective and/or deformed limbs
- Joint contractures
- Joint defects
- Other diseases

The Kimba Cross was specially designed for use with the Kimba seating unit.

Declaration of Conformity

Otto Bock HealthCare GmbH as manufacturer with sole responsibility declares that the Kimba Cross mobility base/pediatric postural positioning system conforms to the requirements of the 93/42/EEC Guidelines.

Service

Service and repairs on the Kimba Cross may only be carried out by Otto Bock healthcare providers. Should any problems arise, please contact the supplier of your Kimba Cross. Any necessary repairs will be made exclusively with authentic Otto Bock spare parts there.

Your authorized Otto Bock healthcare provider:

Safety Instructions

Before using the Kimba Cross, you should become familiar with the handling, function and use of the product.

a) When using the Kimba Cross with rigidly mounted or with swivelling front wheel:

- Get to know in particular how the Kimba Cross with your seating system reacts when the center of gravity is shifted; for example on slopes or inclines or when clearing obstacles, like steps and curbs.
- Use your Kimba Cross properly. For instance, do not drive into obstacles (including steps, curbs) without braking.
- Stairs should be ascended/descended only with assistance from an attendant. If devices such as ramps or elevators are available, they should be used. If such devices are missing, two attendants must carry the Kimba Cross over the obstacle.
- When using lifting platforms, be sure the anti-tipper is outside the danger-area.
- Only lift the Kimba Cross with parts that are solidly attached.
- Park the Kimba Cross only on level surfaces and always apply the drum brakes. Avoid parking the Kimba Cross on an incline.

Make sure that you bring the seat to an upright position.

- You **must** use the anti-tipper accessory whenever the center of gravity is shifted rearwards in the Kimba Cross (such as when using the swivelling front wheel, or when the backrest is in the sleeping position). With the center of gravity shifted rearwards, and without the anti-tipper accessory, the Kimba Cross can tip over, even on a flat surface.
- Do not attach any objects to the pushbar which can affect the center of gravity.
- When your child reaches for objects in front, to the side or behind the Kimba Cross, make sure that he or she does not lean out of it too far, since the shift in the center of gravity might cause the Kimba Cross to tilt or tip over.
- Before leaving the Kimba Cross and before getting into and out of it, always lock the brakes.
- Never leave your child unattended in the Kimba Cross, even if the belts are in use and the brakes are engaged.
- Check the pneumatic tires for proper inflation levels (printed on the outer wall of the tire) and make sure the tires have sufficient tread depth.
- Please keep packaging material away from children. Plastic packaging presents the danger of suffocation.
- Trays must not be loaded with heavy objects.
- The Kimba Cross has been approved for the transport of only one child.
- All accessories and add-on components will reduce the remaining maximal load capacity.
- The maximal load for the Kimba seating unit is 35 kg. The Kimba Cross mobility base has a load capacity of 50 kg.

b) Also, when using the Kimba Cross as a bicycle trailer:

- Never use a tray when using the Kimba Cross as a bicycle trailer.
- The maximal sitting height of the body (measurement from buttocks to head) must not exceed 70 cm.
- Be careful riding the bicycle because the trailer will change how the bicycle handles and increase the stopping distance.
- Please note that when using your Kimba Cross in the public traffic, you must observe the traffic regulations according to the vehicle regulations for your area. For this reason, all components of the trailer accessories package must be mounted on the trailer.
- The child sitting in the trailer must wear seat belts and a safety helmet. A four-point belt and a groin strap must be applied to prevent the child from slipping out of the Kimba Cross.
- This accessories package, not including an additional brake (for use on the trailer) can be used with bicycle trailers with a maximal total weight of 40 kg.
- Modifications on the coupling and other additional assemblies are not allowed. Please observe these instructions, otherwise you can expose your child and yourself to danger.
- **Only use the supplied "Weber" coupling.**
- **Read the Instructions for Use of the "Weber coupling" included with this unit as well as the instructions attached to the draw-bar.**

c) General safety instructions

- **You must check the proper fit and clamping pressure of the axle!**
- Be careful not to injure your fingers during adjustment and assembly.
- Use your Kimba Cross with Otto Bock seating system only in accordance with the instructions for use.
- When using a seating unit from another manufacturer, this manufacturer must confirm compatibility.
- **Every time you mount the wheels, make sure that the rear quick-release axles have engaged correctly and that the front quick-release axle has been locked with sufficient clamping pressure.**
- **After each readjustment of the seat tilt, make sure that you firmly tighten the turning handle. Also check the handle often during use.**
- **Make sure to snugly re-tighten the screws (two on the left and two on the right) after adjusting the telescoping front frame.**
- **We would like to point out that children seated in the seating systems must at all time be secured with suitable safety systems (straps/belts).**
- **Using your Kimba Cross for transportation in motor vehicles.**

We recommend that, wherever and whenever possible, Kimba Cross users transfer to the seats installed in the motor vehicle and use the corresponding vehicle restraint systems, because this is the only way to ensure optimal protection of the passengers in case of an accident. For the time being, Otto Bock has not yet released the Kimba Cross for use as a motor vehicle transportation seat. If you'd like to use it in this way, you must have your vehicle specially outfitted with a restraint system that conforms to any applicable standards. Otto Bock will provide a release upon request.

d) When using the Otto Bock Kimba seating unit:

- The footplate must not be used as a stepstool when getting into or out of the Kimba seating unit.
- Use caution near fire or flammable objects, especially when holding lit cigarettes. The seat and back padding could catch fire.

Unpacking and Setting up

In the original package you will find the following components:

- Frame, folded
- Two rear wheels with quick-release axle
- One front wheel with quick-release lock and mounted splash guard
- Kimba seating unit (if included in the order)
- Instructions for Use and Allen wrench, size 4
- Accessories as ordered

To unfold the frame please proceed as follows:

- Carefully remove the transport safety snaps and packaging material.
Next, remove the locking pin from the frame lugs (Fig. 1).
- Fold up the pushbar (Fig. 2) and position the back brace between the two lugs. The back brace has three bore holes which let you set the pushbar at different angles. Select the appropriate back brace hole for your desired pushbar angle (Fig. 3). Put the locking pin through the selected hole in the back brace and through the lugs and then secure with the clip (Fig. 4). For mounting the rear wheels to the frame, insert the quick-release axles into the rear wheels and press the quick-release axle with the wheel into the axle housing (Fig. 5). **Make sure that the axles have correctly locked in place!** Mounting the front wheel: For mounting the standard, rigid-mount front wheel with pre-assembled splash guard, please proceed as follows: Insert the front wheel with splash guard into the fork and push it to the rear and up (Fig. 6). Close the eccentric lever of the quick-release axle. Make sure the recess in the back of the splash guard engages the crossbrace of the front frame (Fig. 7).

Important: Make sure the axle is properly engaged and has enough clamping pressure!

The clamping pressure can be adjusted by turning the quick-release lever or the counternut.

- To fold the Kimba Cross together, please reverse the directions mentioned above.

Outdoor Mobility Base

Functions:

Pushbar

The pushbar can be adjusted in height and angle to better fit the attendant.

Adjusting the Height of the Pushbar

Loosen the clamping lever on both sides, move the pushbar to the desired height and re-tighten the clamping levers until they are snug (Fig. 13).

Adjusting the Angle of the Pushbar

Flip up the clip of the locking pin and pull out the locking pin to release the back brace. Choose another hole in the back brace to change the pushbar angle. There are three possible positions. Reinsert the locking pin and secure with the clip (Fig. 14).

Brakes: Operation and Adjustment

Pull the large brake levers to apply the drum brakes (Fig. 72, item A). Simultaneously move the small levers downwards (Fig. 72, item B) to lock the brakes. Pull the large brake levers (Fig. 72, item A) again to release the brakes.

You can adjust the drum brakes (independently on the left and right side) to always achieve optimal braking. To do this, loosen the counter nut (Fig. 73, item A) and turn the adjustment nut (Fig. 73, item B) until you reach the braking effect you desire. Snugly re-tighten the counter nut.

Adjusting the Seat Tilt

To change the angle of the seating unit, loosen the turning handle on the left frame side (in driving direction) up to the stop. Next, push in the spring-mounted turning handle to release the adjustment bar. The seat can now be moved freely around the axle of the seat adapter. Bring the seat into the desired position, release the turning handle and wiggle the seat back and forth to re-engage it with the adjustment bar.

Firmly re-tighten the turning handle! (Fig. 15).

Frame Length/Adjusting the Wheelbase

The telescoping front frame lets you use mounted seating units of various sizes, and offers a wheelbase adjustment range between 800 mm to 920 mm. To change the wheelbase, loosen and remove the two screws on either side of the oval tube front frame. Push in or pull out the front section until the desired position has been reached (Fig. 16, 17). **Reposition the screws and snugly re-tighten. Each position must be secured with two screws on each side !**

Outdoor Mobility Base – Accessories (optional)

The Kimba Cross mobility base is designed as a modular assembly system, which means that certain accessories may be adapted to it. The following options and accessories may make it easier to use your Kimba Cross mobility base (Fig. 18).

Swivelling Front Wheel

The optional swivelling front wheel reduces the turning radius of your Kimba Cross, making it easier to use in tight spaces (Fig. 22). First, remove the rigid-mount front wheel and the mounted splash guard in the reverse order as described under "Unpacking and Setting Up".

Next, insert the swivelling front wheel into the fork as shown in Fig. 20 and push it to the rear and up. Make sure that the attachment plate engages with the slot of the crossbrace. Close the eccentric lever of the clamp axle (Fig. 21).

It is very important that you check for the proper fit and clamping pressure of the axle!

Splash Guard for Rear Wheels / Clothing Protectors

To mount these snap-on wheel covers, take off the rear wheels by first removing the quick-release axles (press the central bolts to release the locks and pull the axles out). Pull the rear wheels off the axle housings. Screw the attachment elements for the splash guards to the holes provided for this purpose (Fig. 23, item B). Snap the splash guard into the receiving holes (Fig. 24). Finally reinsert and lock the rear wheels (Fig. 25).

Anti-tipper

The anti-tipper prevents the mobility base from tipping too far backward.

Loosen the screw of the right rear axle and remove the nut and lock washers. Put the anti-tipper on the rear axle while making sure that the fork of the anti-tipper engages with the crossbrace (Fig. 26). Remount the lock washers and nut and re-tighten the nut.

You can bring the anti-tipper from active position (Fig. 27) to passive position (Fig. 28) and vice versa by pressing downwards and then rotating the anti-tipper arm. In both positions the anti-tipper will lock in place.

Storage Bag

Snap fasteners are used to attach the storage bag below the seat: The side of the storage bag with 3 snap fasteners is attached to the rear crossbrace; the side with 2 snap fasteners is attached to the right and left of the base frame.

Spoke Protectors

The spoke protectors are attached to the spokes with mounting elements.

Use of the Kimba Cross as a Bicycle Trailer

Your Kimba Cross can also be used as a bicycle trailer (Fig. 29). The "bicycle package" includes:

- Bicycle draw-bar incl. quick-release clamp axle
- Weber Coupling piece, type E (trailer side)
- Weber Coupling piece, type E (bicycle side)

First remove the front wheel (Fig. 30) in the reverse order as described under "Unpacking and Setting Up".

Insert the bicycle draw-bar and push it to the rear and up.

Make sure that the attachment plate engages with the slot of the crossbrace (Fig. 31, item A). Close the eccentric lever of the clamp axle (Fig. 31, item B). **Check the proper fit and clamping pressure of the axle!**

Mounting the coupling to the bicycle:

Coupling, type E from the Weber company for mounting to the axle of a bicycle.

Besides the coupling the delivery includes two metal locking disks for bicycle hubs with rigid axle (large hole) or with quick-release mechanism (small hole).

- For mounting, remove the wheel nut or quick-release mechanism and position the basis (Fig. 74, item B) on the axle as shown in the drawing.
- Insert the locking disk (Fig. 74, item V) so that the joint piece (Fig. 74, item G) is approximately horizontal.
- Make sure that the teeth of the locking disk properly engage with the basis.
- Firmly tighten axle and coupling with the wheel nut or quick-release mechanism. Check if the wheel is seated securely.
- Position the safety cable (Fig. 74, item S) around the wheel axle and hang in as shown in the drawing.
- **Finally insert the rubber stopple (Fig. 74, item A).**

Note:

If the hub is equipped with a quick-release mechanism please observe the following: First tighten the adjustment nut of the quick-release mechanism until the lever of the quick-release mechanism can be closed firmly and the wheel is seated securely.

Make sure that the adjustment nut of the quick-release mechanism is screwed onto the clamp axle with a minimum of five turns and that the thread is undamaged! Follow the instructions given in the operation manual for your bicycle and for the quick-release mechanism.

Connecting the trailer to the bicycle:

The instructions for connecting the trailer to the bicycle are written on a sticker which is attached to each draw-bar. The trailer draw-bar will additionally be secured to the bicycle frame with a safety strap (Fig. 75).

Equipment package required according to German Motor Vehicle Traffic Regulations (StVZO), Accessories for bicycle trailer (Fig. 32)

Reflectors: Attach the two red reflectors with the tube clamps to the rear crosstube (Fig. 33). (As an alternative to a second red reflector a combination of rear light plus reflector may be included with the Kimba Cross.) Attach the two white reflectors at the front of the Kimba Cross to the deflector angles using adhesive tape (Fig. 34, item A).

The battery-operated **rear light** should also be attached with tube clamp(s) to the rear crosstube (Fig. 33, item A). (As an alternative to a second red reflector a combination of rear light plus reflector may be included with the Kimba Cross.)

Mounting the Roll Bar:

Mount the roll bar as shown in Fig. 35.

Mounting the Signal Pennant:

Loosen the left cap nut of the pushbar bearing (Fig. 36, item A), insert the fork of the fiberglass rod (Fig. 36, item B) behind the washer and snugly re-tighten the cap nut.

Kimba Multifunctional Seating Unit

Depending on the scope of delivery:

Mounting an Otto Bock Kimba multifunction seat:

- Bring the backrest (Fig. 37, item B) in upright position and fix it in the desired position by tightening the clamping lever (Fig. 38, item E). When pulled away from the screw, the clamping lever can be rotated freely, thus enabling you to turn it to a more convenient operating position.
- Slide the back insert (Fig. 38, item B) into the back base (Fig. 38, item D) and clamp it in the desired height by tightening the set screws (Fig. 38, item C).
- We recommend adjusting the seating unit as directed before mounting the padding (please refer to section Adjustment and Adaptation Possibilities of the Seating Unit).
- Place the seat padding (Fig. 39) onto the seat so that the corresponding hook & loop straps are connected. Allow the seat padding to extend approx. 2 cm over the back of the seat edge.
- Adjust the desired depth of the hip support retainers and fix each with the two adjustment screws (Fig. 40).
- Slide the hip support pads over the retainers and attach with hook & loop straps (Fig. 41).
- Put the lateral support pads (Fig. 42, item A) onto the plastic retainers with the padding side towards the inside and secure them with the hook & look closures.
- Pull the back padding (Fig. 38, item A) with its cap over the back insert and close the zipper. Slightly press the padding against the back insert from top to bottom so that the hook & loop closure on the padding (Fig. 42, item B) and back insert closes. Pull the excess back padding down between the seat and lower edge of the back and attach it to the corresponding snap fasteners on the outside of the back (Fig. 38).
- To mount the seating unit to the mobility base (Fig. 8) bring the rear tube retainer of the locking unit over the rear tube of the seat adapter (Fig. 9) and hook the seat in place (Fig. 10). Press the front of the seat edge down until the seat audibly snaps in place (Fig. 11). Make sure it's secure by giving the seat a few short tugs.
- Dismounting the Kimba multifunction seat: While standing behind the seating unit, position yourself on the right side. Hold the head area of the seat back with one hand. Loosen the seat lock by pulling the release lever forward. Tilt the seat to the rear and then pull it up to remove (Fig. 12).

Adjustment and Adaptation Possibilities

Seat depth	Loosen the knurled nuts (Fig. 43, item A) on the underside of the seat and adjust the seat depth by sliding the footrest assembly (Fig. 44, item C). Firmly re-tighten the knurled nuts after adjustment.
Seat width	Loosen the knurled nuts (Fig. 43, item A) on the underside of the seat and adjust the seat width by sliding the hip supports (Fig. 44, item B); abducted settings are also possible. Firmly re-tighten the knurled nuts after adjustment.
Hip support depth	Adjust for the desired depth and fix each support with the two adjustment screws (Fig. 40).
Back height	Loosen the set screws (Fig. 38, item C) with an Allen wrench, move the back insert (Fig. 44, item A) to the desired height, and firmly re-tighten the set screws. From a back height of approx. 55 cm with seat size 1 and of approx. 66 cm with seat size 2, the back plate (Fig. 45) must be shifted from pos. A to pos. B. To adapt the padding accordingly, please refer to section Unpacking and Setting up .

Back inclination	Release the clamping lever (Fig. 38, item E), adjust back inclination, and firmly re-tighten the clamping lever.
Knee angle	Loosen the wing nut (Fig. 46, item A), adjust knee angle, and firmly re-tighten the wing nut.
Footrest height	Loosen the set screws (Fig. 46, item B) with an Allen wrench, adjust the lower leg length, and firmly re-tighten the set screws. The footrest can be flipped up for transport or to support the lower leg.
Footplate angle (option)	The footplate angle can be changed by adjusting the locking bushing (Fig. 47, item A) in the slotted holes. Pull the release strap (Fig. 47, item B) to flip up the footplate.

Kimba Seating Shell Interfaces

Kimba seating shell interfaces (Fig. 48) serve as the basis for individual seating shell fittings.

They are available in various sizes and can include a footrest assembly.

Attachment to the seat adapters and the function and operation of the footrest assembly are identical to those of the seating unit; please refer to section **Kimba Multifunctional Seating Unit**.

Accessories for Kimba Multifunctional Seating Unit

All accessories not installed by the manufacturer must be installed by trained technicians.

The following notes on installation are listed here for technician use.

Before installation of many accessories, the leather strips must be cut open. To do this, first mark the leather. Next pre-punch the start and end of the cut with a hole punch and finally make a cut that connects two holes (Fig. 49).

Straps on accessories are usually extra long to accommodate most positions.

Excess strap ends can be cut off, and the cut edge melted together with a flame (cigarette lighter) in order to prevent fraying.

Roll Loops

To offer better strap positioning in the pelvic area, the Kimba seating unit is delivered with two roll loops. Depending on individual requirements you can mount these roll loops on either side in three different positions (Fig. 50).

Abductor

Make a cut in the synthetic leather strip in the seat padding, place the abductor (Fig. 51) in the desired position, and affix it with screws from below.

When using a slipsack, also cut a slit at the appropriate location.

Models with depth adjustment can be adjusted in the slotted hole.

Armrests (no illustr.)

Snap the armrests into the retainers (Fig. 44) on the seating unit while simultaneously pressing the spring knobs. To remove the armrests, press the spring knobs and pull the armrests upwards off the retainers.

Attention: The armrests must not be used for lifting the seating unit or the rehab stroller/buggy. Armrests cannot be combined with a "**Grabrail**" or a "**Tray**".

Pelvic Band (no illustration)

The pelvic band is used for the fixation and positioning of the pelvis of the patient in the seating system.

For information on mounting and handling please refer to the instructions for use enclosed with the product.

Chest and Shoulder Pad

The chest and shoulder pad straps (Fig. 52, item A) are attached to the back plate with cam-lock buckles and must be attached to the back insert before the back padding is attached to the back plate. To do this, locate the slotted holes beneath the shoulder strap openings on the back insert and use the carriage screws to attach the cam-lock buckles (Fig. 53) to the outside of the back insert. Make sure that the knurled nuts are on the outside.

Guide the shoulder straps for the chest and shoulder pad to the rear. Thread the lower straps between seat and back to the rear, through the roll loop (Fig. 50). Thread the straps through the open cam-lock buckles (Fig. 53) and clamp in position by closing the cam-lock buckles.

Applying the Chest and Shoulder Pad

Before seating the child in the Kimba Cross, release the four cam-lock buckles (Fig. 53) on the back plate and slightly pull the chest and shoulder pad to the front. Then open the cam-lock buckles at the lower end of the pad (Fig. 52, item B), pull out the straps, and position them on the sides of the seat. Place the child in the Kimba Cross and pull the chest and shoulder pad over the child's head onto his or her chest. Next, thread the straps into the lower cam-lock buckles of the pad (Fig. 52, item B) all the way to the stop (rivet) and close the buckles.

Then tighten the lower straps from behind (making sure that the chest and shoulder pad is positioned symmetrically on the pelvis) and close the buckles on the lower straps. Finally, tighten the upper straps on the backside of the seat and close the buckles on the upper straps.

To ensure the correct fit, make sure that the straps are not touching the child and that the reinforced edge of the pad is not in contact with the strap guide slot.

The pad will provide the best upper-body support when the strap guide slot is approx. 2-3 cm above the child's shoulder. If necessary, readjust the back insert accordingly.

Canopy

- First mount the canopy adapters to the back insert (Fig. 54).
- Make sure that the male side of the quick-release buckle is inside the canopy material (Fig. 55).
- Later, the canopy will be connected or separated using only the male side of the quick-release buckle (Fig. 56, item A).
- For angle adjustment of the canopy, adjustable ratchet joints are used (Fig. 56, item B).
- Spread the canopy/ wrap over the seat's backrest.

Canopy with Rain Cover (Fig. 57)

- Take the rain cover out of the net bag attached to the wrap, unfold the rain cover, pull it over the unfolded canopy and put it over the footrest assembly.

The backrest may not recline correctly when the canopy with rain cover is in use.

Fixation Vest

To attach the fixation vest straps, rivet (from inside to outside) the six cam-lock buckles to the outside of the back basis (Fig. 58, item A). Use countersunk head rivets. Thread the shoulder straps of the fixation vest through the openings in the back padding to the rear. The center "axillary straps" must be threaded at the appropriate height between the back frame tube and back plate (Fig. 58, item B).

Thread the lower straps between seat and back to the rear, through the roll loop (Fig. 50). Thread all straps through the open cam-lock buckles and clamp in position by closing the hinged cam.

Opening the fixation vest:

Open the hook & loop strap (Fig. 59, item C) above the zipper. Pull the zipper down and, as needed, open one or more quick-release buckles (Fig. 59, item A) by pressing on their end tabs.

Closing the fixation vest:

Position the child in the seat, close the zipper, hold the vest against the child's chest and close the open quick-release buckles. If required, re-tighten the individual fixation vest straps in the cam-lock buckles.

Belts (Lap belt, Four-point belt, Foot straps)

Lap Belt:

There are two closure versions for mounting and length adjustment.

- a) Length adjustment with hook & loop straps: Open the hook & loop closure of the lap belt and thread the strap ends between seat and back to the rear, through the roll loops (Fig. 50). Attach the two open ends of the hook & loop straps to each other. (This is used to adjust the length of the lap belt – the straps must overlap by at least 15 cm.)
- b) Length adjustment on the belt buckle: Pull the strap completely out of the length adjuster, thread the strap end between the seat and back to the rear, through the roll loops (Fig. 50), and back between seat and back to the front. Next, thread the free strap end through the strap pad, through the length adjuster on the belt buckle and again through the strap pad.

To open the belt buckle, press the marked button (Fig. 60, item B).

To close the belt buckle, slide the metal tab into the housing until you hear the buckle engage.

Four-point Belt:

Attach the lap belt section of the four-point belt as described for the lap belt.

To attach the shoulder straps of the four-point belt, rivet (from inside to outside) two cam-lock buckles to the outside of the back basis (Fig. 58, item A). Use countersunk head rivets.

Thread the shoulder straps of the four-point belt through the openings in the back padding to the rear. Thread both strap ends through the open cam-lock buckles (similar to Fig. 53), and clamp in position by closing the hinged cam. (This is to set the shoulder strap length.)

To open the belt buckle, press the marked button (Fig. 61). To close the belt buckle, join the plastic ends of the shoulder straps and insert the metal tab of the lap belt and put on the belt buckle so that the metal tab locks in place.

Hook & loop foot straps Drill holes from below through the more posterior section of the footplate at the desired positions. Attach the foot straps with the two screws positioned beside one another to the footplate and tighten with knurled nuts below the footplate. Should straps be necessary also over the forefoot, fit them the same way (Fig. 62).

Grab Rail and Grab Rail Padding

First insert the longer part of the grab rail (Fig. 63) into the seat fixture. Make sure that the spring knobs point towards the front. Then insert the shorter end into the second fixture while pressing the spring knob.

Open the grab rail by both pressing the spring knob and pulling the rail up to remove it.

Thread grab rail padding onto the grab rail.

Headrests

Head supports

Attach the pair of head supports (Fig. 64, item C) to the back insert with carriage bolts and knurled nuts. The back padding already features four hollow rivets (Fig. 65, item A) for this purpose. To make installation and adjustments easier, first open the zipper at the top of the back padding.

The head support screws replace the short carriage screws that connect the back frame tube to the back plate.

Once the knurled nuts have been loosened, the supports can be moved in their slotted holes to increase or decrease the amount of head room. To vary the height of the headrest, adjust the back insert (see **Kimba Multifunctional Seating Unit**).

Back insert with integrated headrest

The Kimba multifunctional seating unit can also be equipped with a back insert with integrated headrest (Fig. 66).

The procedure for height adjustment is identical to that of the headrest version described above. The free space for the head can be varied in increments of 1 cm (up to 3 cm on each side) by sliding the supplied hard foam pads under the side padding sections.

Rain Cape

With the zipper open pull the hood over the child's head, then unfold the upper brim over the backrest and the lower brim around the footrest.

Slipsacks (Fig. 67)

Place the summer, winter or sheepskin slipsack in the Kimba Cross. Tie the straps to the lower back section and tie the upper straps to the frame of the back insert. If using an abductor or belts, make a cut in the corresponding synthetic leather strips using a sharp blade. Zippers allow the upper part of the slipsack to be opened or removed.

Groin Strap

When fitting the groin strap for the first time, it is best to remove the seating unit from the outdoor mobility base.

Position the groin strap with the quick-release buckles on the thigh belts pointing downwards on the seat surface.

Thread the attachment straps of the groin strap between seat and back padding, wrap them around the back of the seat edge and affix with the hip support screws (Fig. 68).

Thread the tensioning straps for the thigh belts (the end with a loop) on the right and left side through the roll loops (Fig. 50). Thread the free end through the loop and tighten the tensioning strap.

Now pull the free ends of the tensioning straps toward the seat padding.

- You can now seat the child in the Kimba Cross on the groin strap.

Guide the thigh belts up between the legs and over the thighs to the outside. The part of the thigh belt to which the buckle is attached should lie on the pelvic bone.

- Insert the male side of the quick-release buckles located on the tensioning straps into the buckles (Fig. 69) on the thigh belts.

Tighten the thigh belt by pulling the free end of the strap that comes out of the buckle.

- To disengage the quick-release buckle, press the tab on the plug.

Tray

Snap the tray with the vertical round tubes into the retainers (Fig. 70) on the seating unit while simultaneously pressing the spring knobs. The depth and angle of the tray plate are adjustable.

Attention: The tray can only be used if the seat is mounted in the driving direction and only in combination with a rigidly mounted or swivelling front wheel. A tray may never be used when the Kimba Cross is used as a cycle trailer.

Trays must not be loaded with heavy objects.

Thoracic supports

First determine the correct height position on the backrest for attachment of the thoracic supports (Fig. 45). Mark this point on the synthetic leather strip of the back padding.

Then cut open the synthetic leather strip in the back padding only over the length of the thoracic support holder. Slide the holding plate of the thoracic supports through the incisions under the back padding. Insert the carriage screws of the thoracic supports through the slotted holes in the backrest and screw them to the knurled nuts.

For wider settings it is not necessary to cut open the synthetic leather strips. In this case the holding plates of the thoracic supports can be placed from outside under the back padding (between back padding and back plate).

Loosen the knurled nuts to adjust the supports to reduce or increase the free space for the body and for height adjustment of the supports. It is also possible to adjust the height of the supports by loosening and shifting the Allen head screws (Fig. 45, item A). Finally, attach the pads to the supports and close the hook & loop straps.

Using your Kimba Cross for transportation in Motor Vehicles

We recommend that, wherever and whenever possible, Kimba Cross users transfer to the seats installed in the motor vehicle and use the corresponding vehicle restraint systems, because this is the only way to ensure optimal protection of the passengers in case of an accident. For the time being, Otto Bock has not yet released the Kimba Cross for use as a motor vehicle transportation seat. If you'd like to use it in this way, you must have your vehicle specially outfitted with a restraint system that conforms to any applicable standards. Otto Bock will provide a release upon request.

Instructions for Maintenance and Cleaning

- Check the function of the drum brakes on a regular basis and readjust them if necessary.
- Make sure the cables used to activate the drum brakes can move freely.
- Check the tightness of all screw connections on a regular basis.
- After any adjustments, snugly re-tighten the screws that have been loosened.
- We recommend that you have your Kimba Cross – when used on a normal basis – checked once a year by a Rehabilitation Technology Specialist.
- Clean all frame components and plastic parts using mild detergents only.
- Padding sections can be washed at 40 °C. If washed in a washing machine, put them in a linen bag or a pillow case. In most cases, wiping with a damp cloth is sufficient.
- Do not use your Kimba Cross in salt water, and keep sand or other particles from damaging the wheel bearings.

Warranty

The Otto Bock Warranty applies only when the product is used according to the specified conditions and for the intended purposes, following all manufacturer's recommendations (also see our general terms of sales, delivery and payment).

Technical Data

Measurements are in cm and weights are in kg

Outdoor mobility base

Push handle height	90-115
Total width	66
Total length	140-170
Wheelbase	80-92
Front wheel diameter	40 rigid front wheel 30 swivel. front wheel
Rear wheel diameter	45
Seat tilt	-5° to +30°
Load capacity	50
Weight	15.8
Folding size of outdoor mobility base without seat	94x60x32

Kimba multifunctional seating units 3290 2000

Seat depth	19 - 31
Seat width	20 - 30
Lower leg length	19 - 31
Backrest height	46 - 66
Height of slots for shoulder straps	30
Backrest inclination	0 ° to +90°
Load capacity	35
Weight	7
Folding size (LxWxH)	72x40x22

Seating shell interfaces 3254 8300

Length	33
Width	19
Weight	1

Kimba multifunctional seating units 3293 2000

Seat depth	24 – 40
Seat width	26 - 35
Lower leg length	20 – 37
Backrest height	56 - 70
Height of slots for shoulder straps	35
Backrest inclination	0 ° to +90°
Load capacity	35
Weight	8.8
Folding size (LxWxH)	82 x 44 x 27

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