

ANIOXYDE 1000

INDICATIONS

50 bandelettes de contrôle du taux d'acide peracétique dans la solution reconstituée d'ANIOXYDE 1000.

MODE D'EMPLOI

- Tremper la bandelette pendant une seconde, la sortir en la secouant pour éliminer l'excès de liquide.
- Lecture après 10 secondes.
- Ne jamais interpréter un résultat obtenu plus de 15 secondes après trempage.
- Pour la fréquence d'utilisation, se reporter au mode d'emploi de l'étiquette de l'ANIOXYDE 1000.

CONDITIONS DE STOCKAGE

- Refermer le tube après prélèvement d'une bandelette.
- Conserver à l'abri de l'humidité.
- Respecter la date limite d'utilisation reprise sur l'étiquette.
- Se conserve 90 jours après ouverture.
- Stocker entre +4°C et +30°C.

PRÉCAUTIONS D'EMPLOI

Respectez les précautions d'emploi.
Dispositif médical de classe I.
Avant toute utilisation, lisez l'étiquette et les informations concernant le produit.

INDICATIONS

50 test strips for the level of peracetic acid in the ANIOXYDE 1000 reconstituted solution.

INSTRUCTION FOR USE

- Dip the strip for one second, remove it while shaking it to remove excess liquid.
- Reading after 10 seconds.
- Never interpret a result more than 15 seconds after dipping.
- For the frequency of use, refer to the instructions on the ANIOXYDE 1000 label.

STORAGE CONDITIONS

- Close the container after removing each testing strip.
- Keep away from moisture.
- Respect the expiry date shown on the label.
- Store up to 90 days after opening.
- Store between +4°C and +30°C.

PRECAUTIONS FOR USE

Respect the precautions for use .
Class I medical device .
Before every use, read the label and information concerning the product.



NOIR UNIFORME SUR TOUTE LA PLAGE RÉACTIONNELLE: CONFORME
UNIFORM BLACK ACROSS THE ENTIRE REACTION RANGE: COMPLIANT



COLORATION NON UNIFORME, PRÉSENCE DE POINTS BLANCS,
ÉVANESCENCE RAPIDE DE LA COULEUR: NON CONFORME
NON-UNIFORM COLOURING, EVIDENCE OF WHITE SPOTS,
COLOUR SOON FADES: NON COMPLIANT